Instructions for Use

BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-847-534-6000
1-800-247-3368

IN-237
Rev. 3/21

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

en	Etchant Unit Dose 1,2ml syringes
GENERAL INFORMATION	
Unit dose etchant syringes are designed for use with BISCO's 30ml bulk syringes of phosphoric acid etchants.	
Intended use: licensed dentist.	
PRECAUTIONS: <ul style="list-style-type: none">Read etchant instructions for use prior to filling etchant unit dose 1,2ml syringes.Do not clean, disinfect, or reuse.Use of commonly available protective covering to avoid any contamination of syringe is recommended, if syringe becomes contaminated, discard. Do not dry or disinfect.Instructions for Use available soon request.	
30 ml bulk syringe <ul style="list-style-type: none">Properly connect and secure the bulk syringe to the unit dose syringe.Do not apply force to connect the two systems as it could result in breakage of either syringe connecting Luer.Do not apply force to the back of the piston while syringe is capped, but once unit syringe attached or a withdrawal tip or syringe.If a cap/stop of the unit dose syringe attached, material may squirt back through piston.Always inspect before prior and after use of etching material may squirt from the back of the syringe.Recap bulk syringe while not in use to keep material from drying and getting back into the Luer lock.Do not clean or disinfect etchant as it is the etchant.Use of common available disinfectant between bulk and unit dose syringe as it may cause attack or possible cross-contamination.	
NOTE: Be sure to observe the 30 ml bulk syringe. <ul style="list-style-type: none">Material may be contained between the piston and wall of barrel. Product is volatile caustic.Do not observe that material does not extrude into unit dose syringe, secure unit dose syringe onto bulk syringe system and gently pull back to activate unit dose syringe. Observe unit dose syringe to observe if proper. Use necessary until material begins to extrude into unit dose syringe.	
NOTE: Be sure to observe the unit dose syringe. <ul style="list-style-type: none">Unit dose syringe plunger may pop out back of piston too quickly. Gently pull back on unit dose syringe plunger to activate material.Correct dispensing tip securely, which may otherwise result in leakage or cross-contamination.Single use only. Discontinue use in case of contamination. Discard after use.	
OTHER INFORMATION FOR USE <ul style="list-style-type: none">Always pre-lubricate unit dose (1.2 ml) syringe to the end of the bulk syringe by luring the Luer lock to the end of the syringe opening to the male thread of the bulk syringe system syringe.	

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