



98% overall rating



Bisco
 www.bisco.com

Description

TheraCem Self-Adhesive Resin Cement is a self-etching, self-adhesive, calcium- and fluoride-releasing, radiopaque, dual-cured resin cement. The cement is automixed and dispensed in tips. It requires no etching, priming or bonding of the prepared surfaces and offers a strong bond to zirconia. The cement can be tack cured for 2 to 3 seconds for easy cleanup and removal of excess cement. **TheraCem Self-Adhesive Resin Cement** is available in a natural shade in dual-syringes (8 g) together with mixing, intraoral and root canal tips.

Indications

- Metal and resin crowns, bridges, inlays, onlays (includes porcelain-fused-to-metal, composite-to- metal, resin-based composite/composite-ceramic hybrid)
- Porcelain and ceramic crowns, inlays, and onlays (includes alumina and zirconia)
- Metal (prefabricated or cast) and non-metal/fiber endodontic posts
- Implant-supported restorations
- Orthodontic appliances (brackets)

Unique Features

- Calcium- and fluoride-releasing.
- No preparation of surfaces is required.
- Store at room temperature.
- Automix delivery system.

Consultants' Comments

- "Excellent cement and super easy to use."
- "It tack cured quickly. Clean-up was perfect and there was no sticky residue."
- "The simplicity made it a pleasure to use. I like not needing the steps of etching/ bonding, as well as the product's consistency."
- "Super thin viscosity and film thickness made it easy to dispense and spread along margins."
- "I liked the calcium and fluoride release."
- "No post-operative sensitivity."
- "It was too opaque or creamy for some cases."

"GREAT FOR ZIRCONIA CROWNS. NO EXTRA PRODUCTS TO APPLY OR TO PREP BEFORE CEMENTATION."


Clinical Tips

- Work quickly after placing the material in the restoration.
- Tack cure is fast. Don't tack cure for more than 3 seconds as the cement will become difficult to remove.
- Use a micro brush to distribute the cement.
- Make sure you clean the substrate with alcohol after try-in.


Evaluation Highlights

TheraCem Self-Adhesive Resin Cement was evaluated by 34 consultants, with a total of 596 uses.

- Easy to use.
- Easy clean-up.
- Versatile.
- One-step cementation.
- Radiopaque.

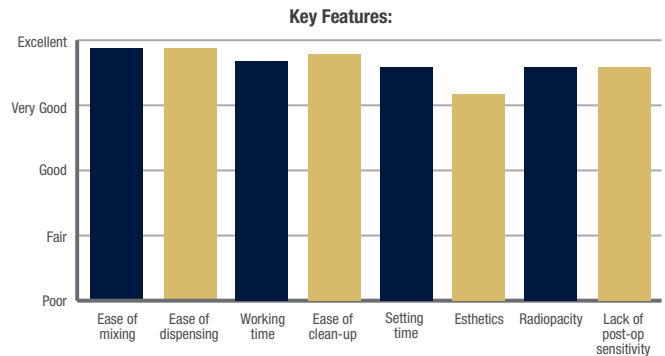

Compared to Competitive Products:

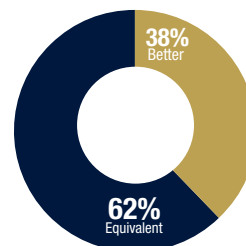

Percentage of Consultants Who Would:

