

INTRAOURAL REPAIR CHECKLIST

- Isolation with a rubber dam
- Prophylaxis of the restorative area
- Mechanical retention
 - Metal/Zirconia = Sandblast or bur roughening
 - Glass Ceramics = HF Etch
- Chemical priming
 - Glass Ceramics: Silane first
 - Metal/Zirconia: MDP containing primer
- Apply D/C opaquer to mask metal (if the metal is exposed) [skip this step for zirconia]
- Apply a porcelain bonding resin or a dental adhesive
- Apply final restorative composite