BISCO


SELECT HV® ETCH

High Viscosity

35% Phosphoric Acid Etchant

SELECT HV® ETCH (35%) **CE 0459**
SELECT HV® ETCH (35%) w/BAC

Instructions for Use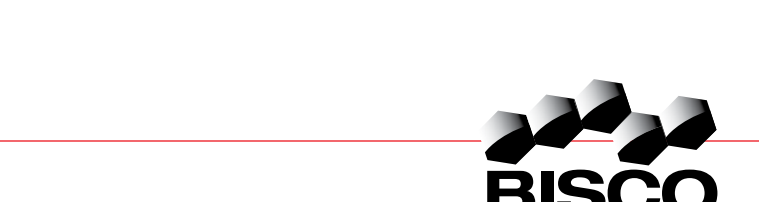

IN-190R14
Rev. 1/21

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

SELECT HV® ETCH 35% Phosphoric Acid Etchant GENERAL INFORMATION

BISCO's new **SELECT HV ETCH** allows for precise selective placement. This etchant provides flexibility, yet is just viscous enough to prevent migration. Selective etching provides a deep phosphoric acid coat on the enamel margins while maintaining the benefits of your self-etch adhesive, such as low postoperative sensitivity and moisture tolerance during the cure. The selective and/or etched surface provides a moisture resistant seal at the enamel margin. The visually brilliant coloring of this new line of etch also makes this phosphoric acid ideal for everyday etching where deep color differentiation is important.

BISCO's **SELECT HV ETCH** is phosphoric acid (H₃PO₄) intended for use on enamel, dentin, and as a cleaning agent for restorative substrates. **SELECT HV ETCH** will effectively remove the smear layer, etch and demineralize enamel and dentin, and remove the smear layer and smear infiltration. The etchant returns glossy clear and quickly, leaving no residue. **SELECT HV ETCH** is also available with benzalkonium chloride (BAC), an antimicrobial agent. Research confirms the development of residual bacterial inhibition zone of up to 10 mm, when challenged (in-vitro) with Actinomyces viscosus. **NOTE:** The inclusion of benzalkonium chloride has not been shown to correlate with a reduction in secondary decay in patients. In-vitro clinical studies to evaluate the effects of benzalkonium chloride on oral bacteria or caries have not been performed.


- Indications For Use:**
- Etching dentin and enamel
 - Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.
- Warnings:**
- The phosphoric acid in the etchant is a severe eye and skin irritant. Injury may result if the etchant is allowed to remain on the skin or mucosa for extended periods of time. In case of contact with other tissues, rinse immediately with plenty of water for several minutes.
 - If accidentally splashed into the eye, flush with copious amounts of water and seek medical attention immediately.
- Caution:**
- Product may contain items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
• Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.
- Precautions:**
- Keep away from direct sunlight. Prolonged exposure to sunlight could cause a discoloration of the etchant; this will not compromise the efficacy of the product.
 - Test the application of Etchant on a mixing pad or a glass slab, BEFORE use on patients so as to become familiar with the pressure required to dispense the Etchant from the tip. Never use intracarily before checking flow from tip. Never use open or plunger to start flow into syringe, this could result in a subsequent uncontrolled discharge of material. If you have any back to suction from the bulk syringe. Disengage unit dose syringe, and release air. Repeat as necessary until material begins to extrude into and out of syringe.
 - Phoghy pastes containing oil or fluoride should not be used as these additives may interfere with etching.
 - Use individual component labels for specific expiration dates.
 - Safety data sheet available at www.bisco.com

- 1. Etching dentin and enamel**
- Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.
- Warnings:**
- The phosphoric acid in the etchant is a severe eye and skin irritant. Injury may result if the etchant is allowed to remain on the skin or mucosa for extended periods of time. In case of contact with other tissues, rinse immediately with plenty of water for several minutes.
 - If accidentally splashed into the eye, flush with copious amounts of water and seek medical attention immediately.
- Caution:**
- Product may contain items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
• Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.
- Precautions:**
- Keep away from direct sunlight. Prolonged exposure to sunlight could cause a discoloration of the etchant; this will not compromise the efficacy of the product.
 - Test the application of Etchant on a mixing pad or a glass slab, BEFORE use on patients so as to become familiar with the pressure required to dispense the Etchant from the tip. Never use intracarily before checking flow from tip. Never use open or plunger to start flow into syringe, this could result in a subsequent uncontrolled discharge of material. If you have any back to suction from the bulk syringe. Disengage unit dose syringe, and release air. Repeat as necessary until material begins to extrude into and out of syringe.
 - Phoghy pastes containing oil or fluoride should not be used as these additives may interfere with etching.
 - Use individual component labels for specific expiration dates.
 - Safety data sheet available at www.bisco.com


SELECT HV 30 ML BULK SYRINGE

- Properly connect and secure the bulk syringe to the unit dose syringe. System will be fully engaged prior to extruding the material.
 - Do not apply force to connect the two systems as it could result in breakage of either syringe connecting Luer.
 - Do not apply force to the back of the piston while syringe is capped, as this could attached or a without tip of syringe.
 - Always use hand and thumb to attach back and front of syringe to back and front of unit dose syringe.
- If bulk syringe is neither capped nor has attached unit dose syringe, material may burst out from tip of syringe.
• Recap bulk syringe while not in use to keep material from drying and getting paste back into box. Do not forcefully place syringe down with tip in the air as material may dry.
• Do not transfer material back and forth between bulk and unit dose syringe as this may cause air pockets and possible cross-contamination.

- NOTE:** Be sure to observe the 30 ml bulk syringe.
- Material may be contained between the piston and wall of barrel. Product is still in unusable condition.
 - If observed that material does not extrude into unit dose syringe, secure unit syringe onto bulk syringe system and apply back to suction to the back of the bulk syringe. Disengage unit dose syringe, and release air. Repeat as necessary until material begins to extrude into unit dose syringe.
- NOTE:** Be sure to observe the Unit Dose Syringe.
- Unit dose syringe plunger may pop out back or forward. Gently pull back on unit dose syringe plunger to extrude material.
 - Control dispensing tip securely, which may pull away too quickly. Keep tip angled on occlusal spatting.
 - Single use only. Otherwise may result in cross-contamination. Discard after use.
1. If using **SELECT HV ETCH** 30 ml bulk syringe system, proceed as follows:
• Attach the pre-attached unit dose syringe to the back of the bulk syringe by turning the Luer lock to the unit dose syringe lightly on to the male thread of the bulk syringe system syringe.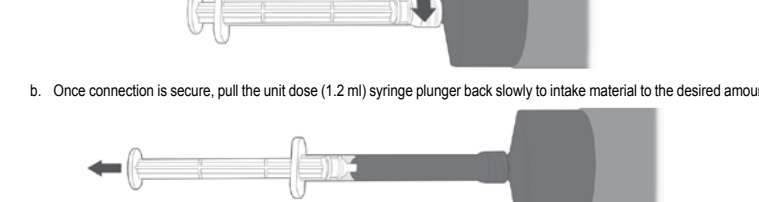

- Once connection is secure, pull the unit dose (1.2 ml) syringe plunger back slowly to intake material to the desired amount.

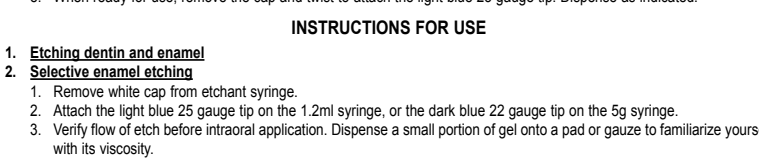- Promptly replace cap back onto the 30 ml syringe.
- If back syringe is not to be used immediately, replace the Luer cap.
- When ready for use, remove the cap and attach the 1.2 ml syringe to the 30 ml 25 gauge tip. Dispense as indicated.

- INSTRUCTIONS FOR USE**
- Etching dentin and enamel**
 - Selective enamel etching**
 - Remove white cap from front of syringe.
Attach the tip (25 gauge) to the unit dose syringe. Attach the back (22 gauge) to the 30 ml 5g syringe.
Verify flow of etch before intracortical application. Dispense a small portion of gel up to a pad or gauze to familiarize yourself with its viscosity.
 - If surface to be etched is not readily wet, clean with a pumice-water slurry. Propoxy pastes containing oil or fluoride should not be used as these additives may interfere with etching. Rinse well.
 - Isolate: Rubber dam is the preferred method of isolation. Use of etching directly onto the surface to be etched. Use an etching enamel in place for 15 seconds.
 - Thoroughly rinse for 5 seconds with suction. If selectively etching enamel, place prior to rinsing to avoid etchant flowing on to dentin.
 - Immediately proceed with the bonding procedure.
 - Discard all single use components.
- Cleaning agent on dental restorative materials**
- SELECT HV ETCH** may be used on restorative substrates to effectively remove organic contaminants. Apply etchant to the surface for 30 seconds and rinse thoroughly with water.
NOTE: BISCO recommends priming the substrate with **PORCELAIN PRIMER**, **BIS-SILANE**™ or **Z-PRIME**™ Plus prior to try-n-and-replace cleaning with phosphoric acid.
HYGIENE: Re-cap syringes with the Luer lock cap. Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended.
- DISPOSAL:** Return to community provisions regarding its use. In their absence, return to national or regional provisions relating to waste.
- WARNING:** BISCO, Inc. recognizes its responsibility to replace products found to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products except as provided. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.
- SELECT HV is a registered trademark of BISCO, Inc.
 - PORCELAIN PRIMER is a trademark of BISCO, Inc.
 - BIS-SILANE and Z-PRIME are trademarks of BISCO, Inc.
 - Dr. Daniel Chan, University of Texas Health Science Center at San Antonio Dental School, Restalr Effect of 1 and 2% Benzalkonium Chloride Incorporated into an Etchant on the Susceptibility of Actinomyces viscosus T143 1993

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