



BisCover_x LV

TECHNIQUE CARD



Manufacturer

BISCO, Inc. • 1100 W. Irving Park Rd. • Schaumburg, IL 60193 • U.S.A. •  847-534-6000 • www.bisco.com

Phone



BISICO France • 120 allée de la Coudoulette • 13680 Lançon de Provence • France •  33-4-90-42-92-92

Phone

REF G-93350K, G-93350K-JP

REF G-93350K-EU: €€0459



Consult
Instructions
For Use

INTC-020R2

Rev. 6/13

BisCover[™] LV *Low Viscosity Liquid Polish*

TECHNIQUE FOR PLACEMENT ON CURED COMPOSITE



Etch for 15 seconds.



Rinse copiously.



Air dry thoroughly.



Apply BISCOVER
LV (one thin coat).



Allow 15 seconds for
evaporation. Do not air thin.



Light-cure 30 seconds
with an LED or halogen light.*

* See instructions for curing times when using other curing lights.



Consult
Instructions
For Use