

*Bisco*

CE

# *NB Instrument Kit*

*Composite Placement Instruments*

## Instructions for Use



BISCO, Inc.  
1100 W. Irving Park Road  
Schaumburg, IL 60193  
U.S.A.  
847-534-6000  
1-800-BIS-DENT

IN-222  
Rev. 9/14

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NB Instrument\* Kit**  
**Composite Placement Instruments**

Inspect instrument for damage prior to use. Do not use if there is any damage.

Clean instrument as soon as possible after use: Instruments can be washed by hand or ultrasonic or automatic washer using a non-corrosive, non-suds, neutral detergent. Do not use abrasives.

Sterilize before first use and between patients: Autoclave 20 minutes at 121°C/250°F (15 psi)

\* NB Instruments are manufactured by BISCO, Inc.