

BISCO

ONE-STEP[®] PLUS *Light-Cured*

Filled Universal Dental Adhesive

Universal-Dental-Adhäsiv, gefüllt

Adhésif dentaire chargé universel

Adhesivo dental universal con relleno

Adesivo dentale universale riempito

Universell dentaladhesiv med filler

Universal dental adhæsiv med filler

Filleriä sisältävä universaali sidosaine

Gevuld universeel tandheelkundig adhesief

Universal dentaladhesiv med filler

Adesivo Dentinário Universal com Carga

Rtcjokkgsijóy paqácomsay le emirvtsijey otriéy

Насыщенный Универсальный Зубной Адгезив

WITH SYRINGE ETCHANT REF U-2260K

CONTENTS:

1 Bottle ONE-STEP PLUS (6ml), 1 Bottle AQUA-PREP[™] F (2ml),
2 Syringes UNI-ETCH[®] w/BAC (5g ea.), Accessories, Instructions



Single Use
Accessories

Rx Only

For Dental
Use Only



Caution



Consult
Instructions
For Use



20°C
68°F
Recommended
Storage Temp.



Keep
Away From
Sunlight
UNI-ETCH



U.S. Patents:

5,749,733 & 5,658,963



BISCO, INC. · 1100 W. Irving Park Rd. · Schaumburg, IL 60193 U.S.A. · 1-847-534-6000 · www.bisco.com

INTC-018R6
Rev. 7/17




ONE-STEP[®] PLUS

Filled Universal Dental Adhesive

BONDING TECHNIQUE



Etch dentin and enamel using UNI-ETCH[®] for 15 sec.

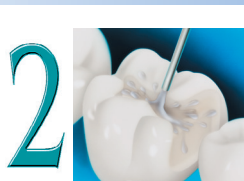

Rinse thoroughly.



Air dry for 2-3 sec.



Apply AQUA-PREP[™] F with absorbent pellet. Allow dwelling for 20-25 sec.



Gently air dry for 1-2 sec. OR Blot dry with absorbent pellet.



At room temperature, shake bottle and briefly vent prior to dispensing. *See instructions for details.



Apply 2 generous coats of ONE-STEP PLUS to the dentin and enamel. Agitate slightly for 10-15 sec.



Air dry gently to evaporate solvent. Leave surface shiny.



Light-cure for 10 sec. per surface.



Consult Instructions For Use

Rx Only