

*Bisco*

# Pro-V<sup>®</sup> IDS KIT

## *Provisional Restorative System*

Provisorisches Restaurationssystem

Système de restauration provisoire

Sistema Restaurador Provisional

Sistema per restauri provvisori

Provisoriskt restaurationssystem

Provisorisk restaureringssystem

Väliaikainen paikkausmenetelmä

Tijdelijk restauratiesysteem

Provisorisk Restaurasjons System

Sistema de Restauração Provisória

Σύστημα για προσωρινές αποκαταστάσεις

Временная Реставрационная Система

## Immediate Dentin Sealing Kit

REF M-10010P

### CONTENTS:

1 ACE<sup>®</sup> ALL-BOND SE<sup>®</sup> Dispenser, 1 ACE ALL-BOND SE Cartridge (2ml), 1 Syringe PRO-V FILL<sup>®</sup> (4g), 1 Syringe AELITEFLO<sup>™</sup> LV (1.5g), 1 Bottle PRO-V COAT<sup>®</sup> (4ml), 1 Bottle CAVITY CLEANSER<sup>™</sup> (7ml), Accessories, Instructions



Single Use  
Accessories



For Dental  
Use Only



Caution



Consult  
Instructions  
For Use



Recommended  
Storage Temp.  
All-Bond SE  
Cartridge Only



Flammable



Irritant



Sensitizer



**BISCO, Inc.**

1100 W. Irving Park Rd.  
Schaumburg, IL 60193  
U.S.A.


847-534-6000




Phone

U.S. Patent: 7,748,980; 8,106,110


## IMMEDIATE DENTIN SEALING USING ACE ALL-BOND SE\*

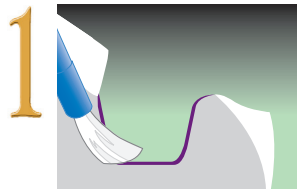


1 Isolate and prep tooth per inlay/onlay preparation guidelines. Apply CAVITY CLEANSER, let dwell for 30 seconds. Suction and blot excess.

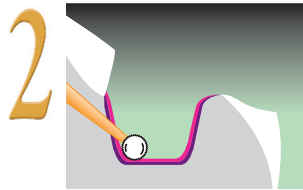


2 Dispense ACE ALL-BOND SE into well and mix until uniformly pink. Apply 1 coat over entire prep, agitating for 10 seconds, dry. Apply 2<sup>nd</sup> coat, agitating for 10 seconds, dry. Light cure for 10 seconds.

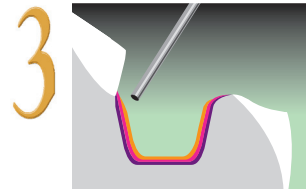


3 Apply a THIN layer of AELITEFLO LV across the pulpal floor and light cure for 20 seconds; if necessary, block proximal wall undercuts with AELITEFLO LV and light cure for an additional 20 seconds.

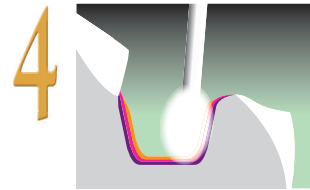


4 Remove air inhibited layer with alcohol pellet.

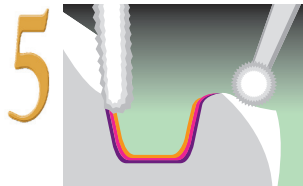

5 Use finishing bur to re-prepare and smooth occlusal enamel margins. Take final impression.

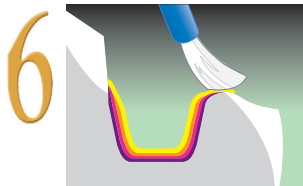

6 Apply PRO-V COAT to lubricate sealed dentin. Gently air dry for 10-15 sec.

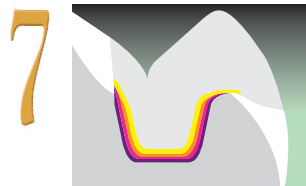

7 Build temporary with PRO-V FILL in 2-3 mm increments. Light cure each increment for 10 seconds.

# Pro-V<sup>®</sup> IDS KIT

\* Refer to instructions for use for detailed technique per indication.