

BisCover™ Liquid Polish

Courtesy of: Mark L. Cannon, DDS, MS

- 1 Isolate teeth.
- 2 Prepare teeth for restoration.
- 3 Apply TYRIAN™ SPE with microbrush. Agitate for 15 seconds.
- 4 Blot with foam pellet.
- 5 Apply 2 generous coats of ONE-STEP® Plus and light-cure for 10 seconds.
- 6 Apply ELITE™ LS in increments curing each increment.
- 7 Develop occlusal anatomy in the last increment and light-cure.
- 8 Place BisCover™ Liquid Polish.
- 9 Light-cure 15 seconds for an instant polish.

NEW from Bisco....BisCover™

- Easily seals and protects in seconds while leaving a dazzling finish on temporary and permanent restorations
- Leaves no sticky air-inhibited layer
- Reduces or eliminates messy and time consuming polishing steps
- Can be used on all types of direct and indirect restorations and appliances
- Increases patient satisfaction
- Ideal for every dental practice



the ultimate finish

BisCover™ by Bisco

1.800.247.3368 • www.bisco.com