

Bisco

CE 0459

TESCERA™ RUBBER SEPARATOR

**Instructions for Use
and
Material Safety Data Sheet
(MSDS)**



Bringing Science to
the *Art* of Dentistry™

BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
847-534-6000
1-800-BIS-DENT

IN-135R2
Rev. 1/10

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

TESCERA™* RUBBER SEPARATOR

GENERAL INFORMATION

TESCERA Rubber Separator is a latex rubber separator, die spacer and blackout material. It is intended for use in a laboratory setting and aids in the creation of dental restorations. Its low viscosity makes it easy to apply and to control its flow. **Rubber Separator's** light pink color facilitates ease in visualizing placement. The set material appears darker than the original color.

Indications for Use:

Laboratory-fabricated restorations.

CAUTION: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

Contraindications:

Not for intraoral use.

Precautions:

- Use With Adequate Ventilation: Rubber Separator has a strong odor and may cause nausea.
- Keep container sealed and from freezing. Freezing temperatures may cause Rubber Separator to polymerize irreversibly.
- Avoid contact with clothing.

INSTRUCTIONS FOR USE

1. Apply **Rubber Separator** to dry model/cast. Increased control in applying the material can be achieved by using an instrument of your choice.

NOTE: Evaporation occurs with each use of **Rubber Separator**. To maintain the fluid-like nature of the material, periodically add a few drops of distilled water to the bottle.

2. Allow to dry. The light pink color of **Rubber Separator** darkens as the material dries.

3. Continue with lab procedure.

* TESCERA is a trademark of BISCO, Inc.

STORAGE: Store at room temperature (20°C/68°F - 25°C/77°F). See individual component labels for specific expiration dates.

WARRANTY: Bisco, Inc. recognizes its responsibility to replace products if proven to be defective. Bisco, Inc. does not accept liability for any damage or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

BISCO, INC.
1100 W. Irving Park Rd.
Schaumburg, IL 60193
U.S.A.
847-534-6000
1-800-BIS-DENT
www.bisco.com

EC Representative
BISICO France
120, allée de la Coudoulette
13680 Lançon De Provence
France
Tél. : 33-4-90-42-92-92