# Bisco

**Light-Cured**

**Low Viscosity Liquid Polish**

**Cures with LED, Halogen and PAC Lights**

## Instructions for Use

**BiscoVest™ LV**  
**Low Viscosity Liquid Polish**  
**GENERAL INFORMATION**

BiscoVest LV is a low-viscosity, light-cured resin formulation used to seal the surface of restorations while leaving a smooth polished surface. Due to its unique proprietary chemistry, BiscoVest LV cures without any sticky oxygen-inhibited layer. BiscoVest LV may reduce or even eliminate the need for manual polishing. The unique chemistry of BiscoVest LV allows for the use of LED, PAC, and halogen curing lights to achieve polymerization.

Indications For Use

BiscoVest LV is used to seal and polish:

- Newly placed direct composites (cured only)
- Previously placed direct composites
- Provisionals
- Resin-modified glass ionomers
- Enamel before or after orthodontic bracket placement
- Enamel after whitening (bleaching)
- SOLiDc-implantation
- F. Infract applications

- Warnings:**
- Highly flammable
  - Avoid splashing into the eyes. If BiscoVest LV comes into contact with the eyes, flush with copious amounts of water and seek medical attention.
  - Upon curing, BiscoVest LV may produce a brief exothermic reaction which is minimized by applying in a thin layer. Do not cure on soft tissue.
  - Use with caution on the chest as it is a flammable liquid and may cause skin irritation. Injury may result if the chest is allowed to remain on the skin or remains in extended periods of time. If accidentally applied to the eye, flush with copious amounts of water and seek medical attention immediately.
  - Use of product on other tissues, misle immediately with plenty of water for several minutes.
  - A rubber dam is recommended for restorations and other indications where contamination may be possible.

**Caution:**

- Cross-contamination: Product may contain items that are designed for use only. Dispose of used or contaminated containers. Do not clean, disinfect or reuse.

- Precautions:**
- Keep UN-ETCH<sup>®</sup> away from direct sunlight.
  - BiscoVest LV is a light-curable material. Use BiscoVest LV immediately once it is dispensed in the mixing well. Prolonged exposure to air and light may lead to an insoluble reaction and blocking of the matrix.
  - To avoid expansion of BiscoVest LV, use containers tightly closed.
  - Avoid contact with the skin. unpolymerized (meth)acrylate resin may cause skin sensitization in susceptible persons. In case of contact, wash immediately with soap and water.
  - Avoid BiscoVest LV skin exposure for 15 seconds prior to curing.
  - Cure completely with a LED, PAC, or Halogen light for 30 seconds.
  - The emitting tip of the curing light should be held at least 2mm away from the rest surface as possible. If a large resin surface is to be light-cured, it is advisable to divide the area into several sections and light cure each section separately.
  - Low curing intensity or insufficient curing will leave an air-inhibited layer on the surface of BiscoVest LV.
  - Appropriate component labels for specific applications exist.
  - Safe use sheet available on request.
  - Safety data sheet available at www.Bisco.com.

- INSTRUCTIONS FOR USE**
- A. On New Placed Composites:**
- Light cure and contour the composite.
  - Apply acid etchant, such as UN-ETCH, to the cured composite and adjacent tooth structure for 15 seconds.
  - Rinse and dry.
  - Continue with Section "Applying BiscoVest LV".
- B. On Old/Previously Placed Composites or for Repolishing:**
- Apply the surface, or sandblast, or roughen with a fine diamond. Rinse and dry.
  - Apply acid etchant, such as UN-ETCH, to the cured composite and adjacent tooth structure for 15 seconds.
  - Rinse and dry.
  - Continue with Section "Applying BiscoVest LV".
- C. Provisionals (all types):**  
**NOTE:** Can be used prior to or after cementation.
  - Contour the provisional restoration.
  - Rinse and dry. (Rinse is not required.)
  - Continue with Section "Applying BiscoVest LV".

**D. On a Light-Cured/Dual-Cured Resin-Modified Glass Ionomer:**

  - Place Resin-Modified Glass Ionomer per manufacturer's instructions.
  - Light cure for 3-5 seconds according to manufacturer's instructions.
  - Continue with Section "Applying BiscoVest LV".
  - Repeat steps 1-3.

- E. In Orthodontic Applications:**
- Prior to Bracket Placement:**
- Clean the surface with a slurry of pumice/water.
  - Etch the surface with orthodontic Etch (20%EtOH - 20CTFFF). Refer to individual component labels for specific application details.
  - Rinse with water (approx. 10-20 seconds) and dry thoroughly. The etched area will appear frosty/white. If not, re-etch for an additional 20 seconds.
  - Continue with Section "Applying BiscoVest LV".
- If Brackets Have Already Been Placed:**
- Follow same instructions as in Section E. "Prior to Bracket Placement", excluding step 5.
- F. Enamel After Whitening (Bleaching):**
- Clean the surface with a slurry of pumice/water.
  - Etch the entire area with 32% hydrogen peroxide on UN-ETCH for 30 seconds.
  - Rinse with water (approx. 10-20 seconds) and dry thoroughly. The etched area will appear frosty/white. If not, re-etch for an additional 20 seconds.
  - Continue with Section "Applying BiscoVest LV".
- G. Acrylic Appliances:**
- Contour the appliance. Clean the appliance thoroughly.
  - Rinse and dry.
  - Continue with Section "Applying BiscoVest LV".
- H. Direct Composites:**  
**NOTE:** Can be used prior to or after cementation.
  - Pumice the surface, or sandblast, or roughen with a fine diamond. Rinse and dry.
  - Apply acid etchant, such as UN-ETCH, to the cured composite and adjacent tooth structure for 15 seconds.
  - Rinse and dry.
  - Continue with Section "Applying BiscoVest LV".

- Applying BiscoVest LV:**
- Dispense BiscoVest LV into a mixing well. Dip the brush into the BiscoVest LV. Wipe excess from the brush onto the side of mixing well.
  - Apply one thin coat of BiscoVest LV in one direction with a smooth stroke. Do not agitate the brush during application. It is very important that one uses the same 15 second curing time for evaporation of solvent after application. Do not air-dry this as this will disperse the material, leaving casting ripples on the surface.
  - BiscoVest LV uses the following curing light and curing times to achieve polymerization. Insufficient curing will leave an air-inhibited layer on the surface of the restoration.
  - A LED Lights:** Light cure for 30 seconds on close range (0-2mm).
  - Halogen Lights:** Using a halogen curing unit for 30 seconds at close range (0-2mm).
  - PAC (Plasma Arc Lamp):** Using a PAC/light cure for 18 seconds at close range (0-2mm).
  - If a cure light is desired, repeat steps 1-3.

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**Observation:** The surface is not tacky but it can be scratched.

- BiscoVest LV requires a minimum curing intensity regardless of the amount of time cured. Holding the light at a distance will reduce the intensity dramatically due to light divergence.
- SOLUTION: When curing a large surface area, it is better to cure smaller sections at close range as opposed to the whole area from a distance.
- The intensity of the curing light is not uniformly distributed as an output falling off in intensity towards the outer periphery. This is further exaggerated as the distance between the probe and surface is increased.
- SOLUTION: When curing a large surface area, it is better to cure smaller sections at close range as opposed to the whole area from a distance.

**DISPOSAL:** Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste restorations and to polymerization (20%EtOH - 20CTFFF). Refer to individual component labels for specific application details.

**WARRANTY:** BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damage to loss of either direct or consequential, stemming from the use of or inability to use of any product as described. Before using, it is the responsibility of the user to determine the suitability of the product for the intended use. The user assumes all risk and liability in connection therewith.

- BiscoVest LV is a trademark of BISCO, Inc.
- UN-ETCH is a registered trademark of BISCO, Inc.

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- Vor dem Setzen der Brackets:**
- Die Phosphorsäure des etchant's in einem sauren und skin irritat. Injury may result if the chest is allowed to remain on the skin or remains in extended periods of time. If accidentally applied to the eye, flush with copious amounts of water and seek medical attention immediately.
  - Use of product on other tissues, misle immediately with plenty of water for several minutes.
  - A rubber dam is recommended for restorations and other indications where contamination may be possible.
- Caution:**
- Cross-contamination: Product may contain items that are designed for use only. Dispose of used or contaminated containers. Do not clean, disinfect or reuse.
- Precautions:**
- Keep UN-ETCH<sup>®</sup> away from direct sunlight.
  - BiscoVest LV is a light-curable material. Use BiscoVest LV immediately once it is dispensed in the mixing well. Prolonged exposure to air and light may lead to an insoluble reaction and blocking of the matrix.
  - To avoid expansion of BiscoVest LV, use containers tightly closed.
  - Avoid contact with the skin. unpolymerized (meth)acrylate resin may cause skin sensitization in susceptible persons. In case of contact, wash immediately with soap and water.
  - Avoid BiscoVest LV skin exposure for 15 seconds prior to curing.
  - Cure completely with a LED, PAC, or Halogen light for 30 seconds.
  - The emitting tip of the curing light should be held at least 2mm away from the rest surface as possible. If a large resin surface is to be light-cured, it is advisable to divide the area into several sections and light cure each section separately.
  - Low curing light intensity or insufficient curing will leave an air-inhibited layer on the surface of BiscoVest LV.
  - Appropriate component labels for specific applications exist.
  - Safe use sheet available on request.
  - Safety data sheet available at www.Bisco.com.

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