

**Bisco**

# Choice<sup>TM</sup> 2

Light-Cured

## Veneer Cement

*Veneer Zement*

*Ciment pour facette*

*Cemento para Veneer*

*Cemento per faccette*

*Fasadcement*

*Veneer/facade cement*

*Laminaattisementti*

*Veneercement*

*Sementering av laminater*

*Cimento para Facetas*

КпнЯБ гйб ьшейт

Цемент Для Облицовок

### STARTER KIT REF C-3900K

#### CONTENTS:

2 Syringes CHOICE 2 Shade: TRANSLUCENT (4.5g ea.), 1 Syringe CHOICE 2 Shade: A1 (4.5g), 1 Syringe CHOICE 2 Shade: B1 (4.5g), 1 Syringe CHOICE 2 Shade: MILKY BRIGHT (4.5g), 1 Syringe UNI-ETCH<sup>®</sup> w/BAC (5g), 1 Syringe 9.5% Porcelain Etchant (5g), 1 Bottle each Bis-Silane<sup>™</sup> Parts A & B (6ml ea.), 1 Bottle each ALL-BOND 3<sup>®</sup> Parts A & B (6ml ea.), 1 Bottle Porcelain Bonding Resin (6ml), Accessories, Instructions



Manufacturer

**BISCO, Inc.**  
1100 W. Irving Park Rd.  
Schaumburg, IL 60193  
U.S.A.  
1-847-534-6000  
[www.bisco.com](http://www.bisco.com)

INTC-024F9  
Rev. 8/18



Single Use  
Accessories

**Rx Only**

For Dental  
Use Only



Caution



Consult  
Instructions  
For Use



Recommended  
Storage Temp.



Keep Away  
From Sunlight  
UNI-ETCH





# Choice™ 2 PORCELAIN VENEER RESTORATION PREPARATION & CEMENTATION


## PORCELAIN VENEER RESTORATION PREPARATION

1




If the veneer has not been etched by the laboratory, etch with 9.5% PORCELAIN ETCHANT for 90 seconds.

2

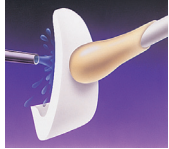

Rinse with water.

3

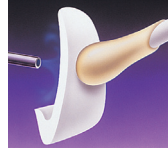

Thoroughly air dry.

4



Mix BIS-SILANE™ Parts A & B and apply 1-2 coats to the internal surface. Wait for 30 seconds.


5



Dry with air syringe.


## TOOTH PREPARATION FOR PORCELAIN VENEER

1

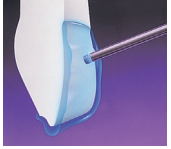


Clean the preparation with a slurry of pumice. Rinse and dry. Etch dentin and enamel using UNI-ETCH® for 15 seconds.

2

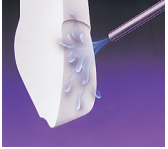

Rinse thoroughly.

3

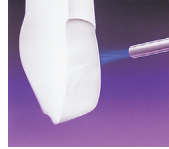

Lightly air dry. Leave preparation visibly moist.


4



Mix ALL-BOND 3® Parts A & B and apply 1-2 consecutive coats to the dentin and enamel.


Note: Additional coats may be required if a lot of dentin is exposed.

5

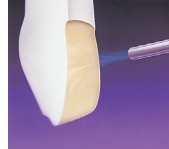

Air dry for a minimum of 10 seconds to evaporate solvent.


6

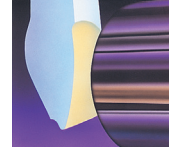

Light cure for 10 seconds.

## PORCELAIN VENEER CEMENTATION

1

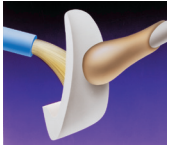


Apply a very thin layer of PORCELAIN BONDING RESIN to the inside of the veneer. **DO NOT LIGHT CURE.**

2




Line inside of veneer with CHOICE™ 2 Veneer Cement.

3

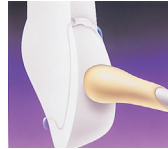

Working in pairs, gently seat one veneer at a time.

4

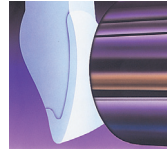

Light cure each for 3-5 seconds to tack the veneers into place. Remove excess cement. Light cure each veneer for 40 seconds from lingual and 40 seconds from the facial.

# Choice™ 2 Veneer Cement

Rx Only

For Dental Use Only



Consult Instructions For Use