

Bisco

Choice™ 2

Light-Cured

Veneer Cement

Veneer Zement

Ciment pour facette

Cemento para Veneer

Cemento per faccette

Fasadcement

Veneer/facade cement

Laminaattisementti

Veneercement

Sementering av laminater

Cimento para Facetas

КпнЯБ гйб ьшейт

Цемент Для Облицовок

STARTER KIT REF C-39010K

CONTENTS:

2 Syringes CHOICE 2 Shade: TRANSLUCENT (4.5g ea.), 1 Syringe CHOICE 2 Shade: A1 (4.5g), 1 Syringe CHOICE 2 Shade: B1 (4.5g), 1 Syringe CHOICE 2 Shade: MILKY BRIGHT (4.5g), 1 Syringe 32% UNI-ETCH® (w/BAC) (5g), 1 Syringe 9.5% Porcelain Etchant (5g), 1 Bottle each Bis-Silane™ Parts A & B (6ml ea.), 1 Bottle ALL-BOND UNIVERSAL® (6ml), 1 Bottle Porcelain Bonding Resin (6ml), Accessories, Instructions



 **BISCO, Inc.**
Manufacturer **1100 W. Irving Park Rd.**
Schaumburg, IL 60193
U.S.A.
 **847-534-6000**
www.bisco.com

INTC-052F1
Rev. 4/15



Single Use
Accessories



For Dental
Use Only



Caution



Consult
Instructions
For Use



Recommended
Storage Temp.



Keep Away
From Sunlight
UNI-ETCH



Toxic



Corrosive



Flammable



Irritant



Sensitizer



Choice™ 2 PORCELAIN VENEER RESTORATION PREPARATION & CEMENTATION


PORCELAIN VENEER RESTORATION PREPARATION

1



If the veneer has not been etched by the laboratory, etch with 9.5% PORCELAIN ETCHANT for 90 seconds.

2

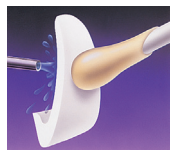


Rinse with water.

3



Thoroughly air dry.

4

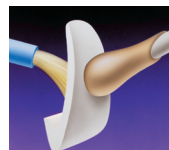

Mix BIS-SILANE™ Parts A & B and apply 1-2 coats to the internal surface. Wait for 30 seconds.


5



Dry with air syringe.


TOOTH PREPARATION FOR PORCELAIN VENEER

1

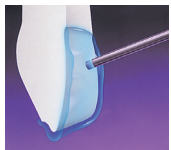


Clean the preparation with a slurry of pumice. Rinse and dry. Etch dentin and enamel using UNI-ETCH® for 15 seconds.

2

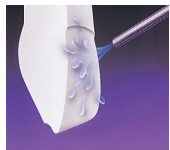


Rinse thoroughly.

3

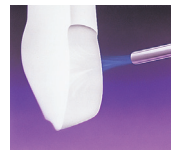


Lightly air dry. Leave preparation visibly moist.

4

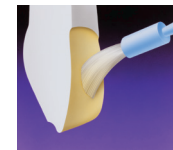


Apply two separate coats of ALL-BOND UNIVERSAL®, scrubbing the preparation with a microbrush for 10-15 seconds per coat. Do not light cure between coats.

5

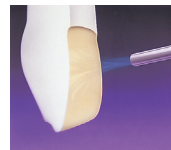

Air dry for a minimum of 10 seconds to evaporate solvent.


6

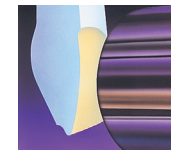

Light cure for 10 seconds.

PORCELAIN VENEER CEMENTATION

1

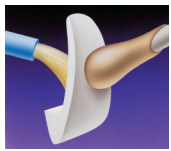

Apply a very thin layer of PORCELAIN BONDING RESIN to the inside of the veneer. **DO NOT LIGHT CURE.**

2




Line inside of veneer with CHOICE™ 2 Veneer Cement.

3



Working in pairs, gently seat one veneer at a time.

4

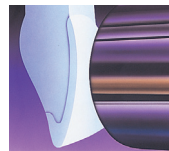

Light cure each for 3-5 seconds to tack the veneers into place. Remove excess cement. Light cure each veneer for 40 seconds from lingual and 40 seconds from the facial.

Choice™ 2 Veneer Cement

Rx Only

For Dental
Use Only



Consult
Instructions
For Use