

**BISCO**

# Choice™ 2

**Light-Cured**

## Veneer Cement

*Veneer Zement*  
*Ciment pour facette*  
*Cemento para Veneer*  
*Cemento per faccette*  
*Fasadcement*  
*Veneer/facade cement*  
*Laminaattisementti*  
*Veneercement*  
*Sementering av laminater*  
*Cimento para Facetas*  
Кпняб гйб ьшейт  
Цемент Для Облицовок

**STARTER KIT**  
**REF C-39010K**

**CONTENTS:**

2 Syringes CHOICE 2 Shade: TRANSLUCENT (4.5g ea.), 1 Syringe CHOICE 2 Shade: A1 (4.5g), 1 Syringe CHOICE 2 Shade: B1 (4.5g), 1 Syringe CHOICE 2 Shade: MILKY BRIGHT (4.5g), 1 Syringe 32% UNI-ETCH® (w/BAC) (5g), 1 Syringe 9.5% PORCELAIN ETCHANT (5g), 1 Bottle each BIS-SILANE™ Parts A & B (6ml ea.), 1 Bottle ALL-BOND UNIVERSAL® (6ml), 1 Bottle PORCELAIN BONDING RESIN (6ml), Accessories, Instructions



 **BISCO, Inc.**  
1100 W. Irving Park Rd.  
Schaumburg, IL 60193  
U.S.A.  
1-847-534-6000  
[www.bisco.com](http://www.bisco.com)

INTC-052F3  
Rev. 10/20



Accessories



UNI-ETCH



<https://www.bisco.com/symbols-glossary/>



# Choice™ 2 PORCELAIN VENEER RESTORATION PREPARATION & CEMENTATION

## PORCELAIN VENEER RESTORATION PREPARATION

1



If the veneer has not been etched by the laboratory, etch with 9.5% PORCELAIN ETCHANT for 90 seconds.

2



Rinse with water.

3



Thoroughly air dry.

4



Mix BIS-SILANE™ Parts A & B and apply 1-2 coats to the internal surface. Wait for 30 seconds.


5



Dry with air syringe.

## TOOTH PREPARATION FOR PORCELAIN VENEER

1

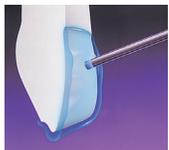

Clean the preparation with a slurry of pumice. Rinse and dry. Etch dentin and enamel using UNI-ETCH® for 15 seconds.

2



Rinse thoroughly.

3




Lightly air dry. Leave preparation visibly moist.

4




Apply two separate coats of ALL-BOND UNIVERSAL®, scrubbing the preparation with a microbrush for 10-15 seconds per coat. Do not light cure between coats.

5

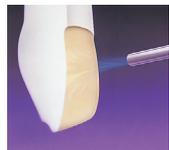

Air dry for a minimum of 10 seconds to evaporate solvent.


6

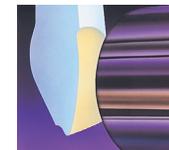

Light cure for 10 seconds.

## PORCELAIN VENEER CEMENTATION

1

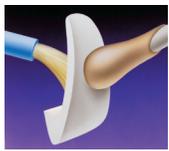

Apply a very thin layer of PORCELAIN BONDING RESIN to the inside of the veneer. **DO NOT LIGHT CURE.**

2




Line inside of veneer with CHOICE™ 2 Veneer Cement.

3



Working in pairs, gently seat one veneer at a time.

4

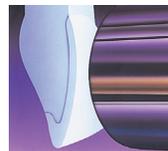

Light cure each for 3-5 seconds to tack the veneers into place. Remove excess cement. Light cure each veneer for 40 seconds from lingual and 40 seconds from the facial.

# Choice™ 2 Veneer Cement

Rx Only

