# FORTIFY™ Light-Cured Composite Surface Sealant

## Instructions for Use

BISCO, Inc.  
1100 W. Irving Park Road  
Schaumburg, IL 60193  
U.S.A.  
1-847-534-6000  
1-800-247-3368

IN-012R10  
Rev. 2/22

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

### GENERAL INFORMATION

**FORTIFY** is a light-cured, low viscosity, unfilled resin formulation.

**FORTIFY** is intended for use as a composite surface sealant. Contouring and finishing of posterior composite restorations with rotary instruments can create microcracks and surface irregularities that contribute to composite wear during the first six months of function. **FORTIFY** is a surface sealant that is formulated with an unfilled resin. A composite surface sealed with **FORTIFY**, has improved wear resistance and long-term marginal integrity. **FORTIFY** is effective with all anterior/posterior composites (Bis-GMA, or UDMA resin base), resulting in enhanced wear properties. Due to the unique formulation, **FORTIFY** can be air-thinned significantly, then light-cured, without interfering with the occlusion.

The principle use of **FORTIFY** is to penetrate and seal the surface of:

1. Direct Composite Restorations
2. Indirect Composite Restorations
3. Provisional Restorations
4. Amalgam Restorations

#### Warnings:

- Avoid splashing into the eyes. If product comes into contact with the eyes, flush with copious amounts of water and seek medical attention.

#### Cautions:

- Cross-contamination: Product may contain items that are designed for one time use. Dispose of used or contaminated unit-dose components and accessories. Do not clean, disinfect or reuse.

#### Precautions:

- Unpolymerized resins may cause skin sensitization in susceptible persons. Avoid contact with the skin. In case of contact with the skin, wash thoroughly with soap and water.
- Refer to individual component labels for specific expiration dates.
- Safety data sheet available on request.
- Safety data sheet available at [www.bisco.com](http://www.bisco.com).

### INSTRUCTIONS FOR USE

**1. For Newly Placed Restorations:** Following adjustment, finishing and polishing of the final restoration, rinse the tooth and restoration with copious amounts of water to remove all debris. **OR**

**For Previously Placed Restorations:** Clean the entire surface with a slurry of pumice and water.

2. Etch the surface of the restoration and approximately 1-2mm beyond the tooth/ restoration margin using an etchant according to manufacturer's instructions. For full coverage restorations, etch the entire surface of the restoration.

3. Rinse with copious amounts of water and thoroughly air dry the tooth and restoration.

4. Place a small amount of **FORTIFY** sealant in a mixing well. Apply a thin layer to previously etched surfaces using a scrubbing motion. Air-thin by blowing a gentle stream of air over this layer to assure an even distribution.

5. Light cure sealant for 10 seconds. Verify occlusion.

**DISPOSAL:** Express through a mixing tip or collect mixed excess material onto a mixing pad. Light cure excess material in sections until fully cured. Dispose of syringe, tips, and cured material according to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

**STORAGE:** Store at room temperature (20°C/68°F - 25°C/77°F).

**WARRANTY:** BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

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\* FORTIFY is a trademark of BISCO, Inc.
\* FORTIFY is een handelsmerk van BISCO, Inc.