

TheraCal PT™


Dual-Cured Resin-Modified Calcium
Silicate *Pulpotomy Treatment*

Rx Only




Consult
Instructions
For Use

1

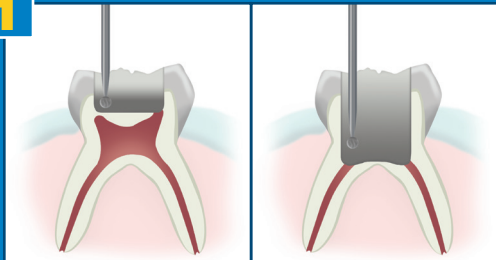


Perform pulpotomy treatment.

2

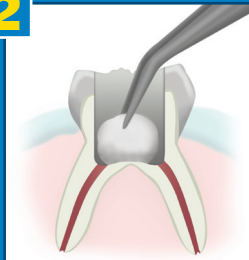


Achieve hemostasis.

3

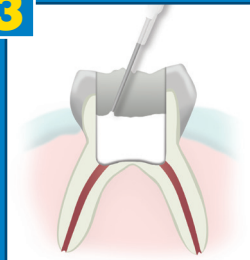

Place TheraCal PT directly in
the pulp chamber, and ensure
good adaptation to the cavity
walls and margins.

4

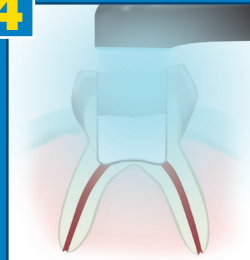

Light cure TheraCal PT for
10 seconds. Place desired
adhesive, base, and/or restora-
tion following manufacturer's
directions for restoring tooth.

BISCO

Dual-Cured

TheraCal **PT**TM

*Dual-Cured Resin-Modified Calcium
Silicate **Pulpotomy Treatment***



TECHNIQUE CARD

 Manufacturer BISCO, Inc. • 1100 W. Irving Park Road • Schaumburg, IL 60193 • U.S.A.
1-847-534-6000 • www.bisco.com

INTC-058R1
Rev. 3/19

Rx Only

