

Bisco

CE 0459

Z-PRIME™ plus 

Zirconia - Alumina - Metal Primer

Zirkoniumoxid-Aluminiumoxid-Metall Primer

Primer Zircono-Alumine-Métal

Primer para Zirconio-Aluminio-Metal

Primer per metallo allumina e zirconia

Zirkoniumdioxid- Aluminiumoxid- metallprimer

Zirkonium-aluminium-metal primer

Zirkonia-Aluminioksidi-Metalli Primeri

primer voor zirkonium, alumina en metaal

Zirconia-Alumina-Metall Primer

Primer Zircónio-Alumina-Metal

Primer για ζirkonία, αλουμίνα και μεταλλικούς σκελετούς

Цирконий-Окись Аллюминия-Метал Праймер

Primer do Cyrkonii – Aluminium – Metalu

INST-070R8

REF B-6002P

Contents: 1 Bottle (2ml), Instructions



For Dental
Use Only



Caution



Consult
Instructions
For Use



Recommended
Storage Temp.



BISCO, Inc.
1100 W. Irving Park Rd.
Schaumburg, IL U.S.A.
1-847-534-6000
www.bisco.com




BISICO France
120 allée de la Coudoulette
13680 Lançon de Provence
France
33-4-90-42-92-92



U.S. Patent: 8,785,518

Z-PRIME™ plus+

1

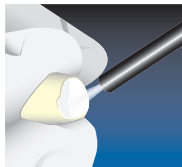

Surface can be cleaned using one or more of the following options: pumice scrub, ultrasonic cleaning with alcohol or acetone, steam cleaning, and/or sand blasting using aluminum oxide (30-100 micron grit at 30-45 psi). Rinse and air dry.

2



Apply 1-2 coats of Z-PRIME Plus, uniformly wetting the internal surface. Dry with an air syringe for 3-5 seconds.

3



Proceed with cementation using a luting cement (e.g. DUO-LINK UNIVERSAL™).



Consult
Instructions
For Use