


# Quantium™

## Universal Composite

### Direct and Indirect Tooth Preparation Treatment


*Proper isolation is recommended for all direct restorations and other indications where contamination may be possible.*

1

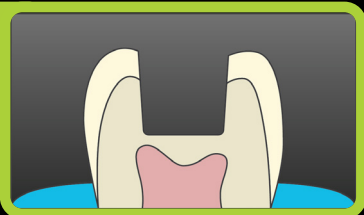


Select the desired Quantum shade prior to isolating and preparing the tooth. Clean, rinse and isolate.

2

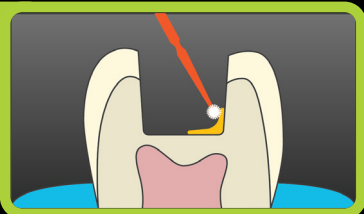


If needed, place matrix, and apply an adhesive, such as All-Bond Universal, following manufacturer's instructions.

3

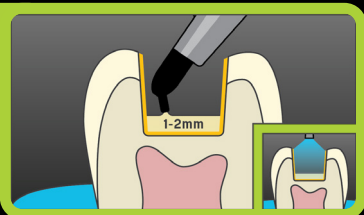


Dispense desired amount of Quantum body shade onto a mixing pad by turning the syringe handle clockwise. If using unit dose tips, inject Quantum directly into preparation. Place Quantum in 1-2 mm increments into the cavity preparation up to the dentin enamel junction, leave space for a final enamel layer. Light-cure each increment with a curing light for 20 seconds.

4

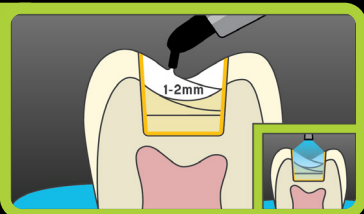

Select the desired Quantum enamel shade (optional) for the last 1-2 mm increment of the restoration, apply, and light cure each increment for 20 seconds.

5

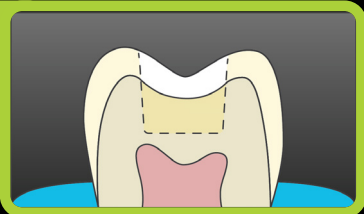

Check the occlusion, finish, and polish.