
Bisco

REVEAL™ Clear Matrix

Clear VPS Impression Material

Self-Cured

Instructions for Use

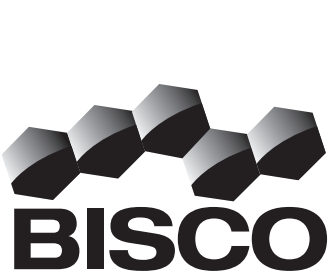BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-847-534-6000
1-800-247-3368

IN-213R4
Rev. 8/17

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

REVEAL™* CLEAR MATRIX Clear VPS Impression Material
GENERAL INFORMATION
BISCO's REVEAL Clear Matrix is a clear vinyl polysiloxane impression material that reproduces fine detail for the fabrication of provisional restorations. Take the preliminary impression with this clear material and use the matrix for intra-oral and extra-oral polymerization of provisional materials.
Indications for Use:
• Use for intra-oral polymerization of light activated provisional materials.
• Use as a general impression material.
Warnings:
• If accidentally splashed into the eye, flush with copious amounts of water and seek medical attention immediately. In case of contact with other tissues, rinse immediately with plenty of water for several minutes.
Cautions:
• Cross-contamination: Product may contain items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
Precautions:
• Refer to individual component labels for specific expiration dates.
• Store with mixing tip in place to prevent hardening of impression material at cartridge openings
• Powder from gloves can impair set.
• Safety data sheet available on request.
• Safety data sheet available at www.Bisco.com.

REVEAL™* CLEAR MATRIX Clear VPS Impression Material
GENERAL INFORMATION
BISCO's REVEAL Clear Matrix is a clear vinyl polysiloxane impression material that reproduces fine detail for the fabrication of provisional restorations. Take the preliminary impression with this clear material and use the matrix for intra-oral and extra-oral polymerization of provisional materials.
Indications for Use:
• Use for intra-oral polymerization of light activated provisional materials.
• Use as a general impression material.
Warnings:
• If accidentally splashed into the eye, flush with copious amounts of water and seek medical attention immediately. In case of contact with other tissues, rinse immediately with plenty of water for several minutes.
Cautions:
• Cross-contamination: Product may contain items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
Precautions:
• Refer to individual component labels for specific expiration dates.
• Store with mixing tip in place to prevent hardening of impression material at cartridge openings
• Powder from gloves can impair set.
• Safety data sheet available on request.
• Safety data sheet available at www.Bisco.com.
TECHNICAL INFORMATION
Working Time: 1 minutes 30 seconds at room temperature (20°C/68°F - 25°C/77°F)
Oral Setting Time: ≤ 2 minutes 30 seconds
Working Time and Setting Time may vary based on storage conditions, temperature, humidity, etc.
INSTRUCTIONS FOR USE
1. Insert cartridge into gun, remove twist off cap, and extrude about 1/4 inch of material, while checking for even flow. Discard the dispensed material and wipe end of cartridge clean.
2. Attach the auto-mix tip and squeeze the cartridge handle with smooth, even pressure.
3. Do not remove the auto-mix tip after use. The used tip serves as a convenient seal until next use.
4. Select a rigid, clear plastic tray sized to provide at least a 2-3mm thickness of REVEAL Clear Matrix .
5. Brush a thin layer of tray adhesive onto the tray and allow to air dry.
6. Inject REVEAL Clear Matrix into the tray. Working time is 1 minute 30 seconds.
7. Seat tray.
8. Remove REVEAL Clear Matrix after 2 minutes 30 seconds in the mouth. Rinse the impression under cold water and dry.

DISPOSAL: Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

STORAGE: Store at room temperature (20°C/68°F - 25°C/77°F). See individual component labels for specific expiration dates.

WARRANTY: Bisco, Inc. recognizes its responsibility to replace products if proven to be defective. Bisco, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

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* REVEAL is a trademark of BISCO, Inc.
* REVEAL é uma marca comercial da BISCO, Inc.