

Bisco

CE 0459

AQUA-PREP™ F

Fluoridated Rewetting Agent

Instructions for Use

Imported by:
BISCO Dental Products Canada
2571 Smith Street
Richmond, BC V6X 2J1
Canada
1-604-276-8662

IN-037-15R3
Rev. 7/17



BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-847-534-6000
1-800-247-3368

AQUA-PREP™* F

Fluoridated Rewetting Agent

GENERAL INFORMATION

AQUA-PREP F, an aqueous solution of fluoride and hydroxyethyl methacrylate (HEMA), is a rewetting agent to be used prior to the application of total-etch adhesive systems. **AQUA-PREP F** resolves the often asked question, "How wet is wet?" Application of **AQUA-PREP F** will provide an ideal amount of moisture necessary for bonding agents.

Indications For Use:

Rewetting and desensitizing prior to application of a total-etch adhesive.

Warning:

- Avoid splashing into the eyes. If **AQUA-PREP F** comes into contact with the eyes, flush with copious amounts of water and seek medical attention.

Precaution:

- Product may cause skin sensitization in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.
- See individual component labels for specific expiration dates.
- Safety data sheet available on request.
- Safety data sheet available at www.Bisco.com.

INSTRUCTIONS FOR USE

1. Etch dentin and enamel for 15 seconds with a phosphoric acid etchant such as ETCH-37™*, SELECT HV®* ETCH or UNI-ETCH®*. Rinse thoroughly for approximately 5 seconds.
2. Air dry for 2-4 seconds or blot with an absorbant pellet. A dry, *but not desiccated*, surface is ideal before applying **AQUA-PREP F**. Dentin should appear dull, and etched enamel should appear frosted.
3. Dispense 2 drops of **AQUA-PREP F** into a mixing well. With a brush, apply to the dentin and enamel surfaces. Do not apply with a cotton pellet.
4. Allow **AQUA-PREP F** to soak for 20 seconds.
5. Gently air dry or blot with an absorbent pellet to remove excess **AQUA-PREP F**. Avoid pooling, especially at the internal line angles of the preparation. The resulting surface should have a shiny appearance.
6. Apply adhesive resin system such as ALL-BOND 2®*, ALL-BOND 3®*, ACE®* ALL-BOND TE™* or ONE-STEP®* ONE-STEP PLUS ®* per manufacturer's instructions.

DISPOSAL: Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

STORAGE: Store at room temperature (20°C/68°F - 25°C/77°F). See individual component labels for specific expiration dates.

WARRANTY: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

* AQUA-PREP, ETCH-37 and ALL-BOND TE are trademarks of BISCO, Inc.
SELECT HV, UNI-ETCH, ALL-BOND 2, ALL-BOND 3, ACE, ONE-STEP, and ONE-STEP PLUS are registered trademarks of BISCO, Inc.

BISCO, INC.
1100 W. Irving Park Rd.
Schaumburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com

 BISCO France
120, allée de la Coudoulette
13680 Lançon De Provence
France
33-4-90-42-92-92