
Instructions for Use

BISCO, Inc.
1100 W. Irving Park Road
Schamburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com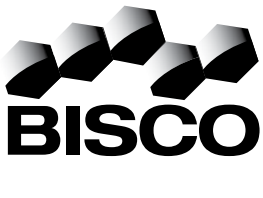

BISICO France
208, allée de la Coudoulette
13680 Lançon de Provence
France
33-4-90-42-92-92

IN-176R10
Rev. 10/20

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

en CORE-FLO™ DC

Dual-Cured Flowable Core Build-Up Composite

GENERAL INFORMATION

CORE-FLO DC is a flowable dual-cured, radiopaque core build-up composite. CORE-FLO DC is specifically designed to cement posts and build-up the core structure in one easy application. CORE-FLO DC is also an ideal dentin replacement material. CORE-FLO DC comes in an automix dual-syringe and is dispersed with the DUAL-SYRINGE DISPENSER. CORE-FLO DC is intended for use with a bonding agent that is compatible with dual-cured materials, such as BISCO adhesives.

Indications for use:

The principle indications for use of CORE-FLO DC are:

1. Core build-up material
2. Cementation of posts or pins
3. Dentin replacement material

Intended user: licensed dentist

Warnings:

- Contamination by saliva, blood, or water should be avoided during these procedures. When using dental adhesives, contamination will compromise dentin bonding and may result in decreased longevity of the restoration.
- A rubber dam is recommended for all indications where contamination may be possible.

Cautions:

- Cross-contamination:
 - Product may contain items that are designed for one-time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
 - Use of commonly available hygienic protective covering after any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Precautions:

- Avoid contact with the skin. unpolymerized (meth)acrylate resins may cause skin sensitization in susceptible persons. In case of contact, wash skin with soap and water.
- To prevent polymerization or clogging, the dual-syringe mixing tip should be left in place until the next application.
- If refrigerated, allow CORE-FLO DC to reach room temperature before use.
- Individual component labels for specific expiration dates.
- Safety data sheet available on request.
- For further information, visit www.bisco.com.
- Instructions for Use available upon request.
- Instructions for Use available at www.bisco.com.

TECHNICAL INFORMATION

At room temperature (20°C/68°F - 25°C/77°F)

Working Time: Minimum 2 minutes

Setting Time: Maximum 7 minutes

Working time and Setting Time may vary based on storage conditions, temperature, humidity, etc.

INSTRUCTIONS FOR USE

CORE MATERIAL MIXING PROCEDURES:

1. Insert the syringe into a Dual-Syringe Dispenser and remove the syringe cap or mixing tip from the dual-syringe.
2. Dispense a small amount of material onto a mixing pad to eliminate any voids in the core of the dual-syringe.
3. Attach a mixing tip to the dual-syringe by lining up the key and keyway. Then turn the brown locking mechanism clockwise.

4. Dispense the desired amount of material by using gentle steady pressure. (See TECHNICAL INFORMATION Section on Working and Setting Time information.)

1. CORE BUILD-UP USING CORE-FLO DC

- (1) Isolate the tooth.
- (2) Apply an adhesive according to the manufacturer's instructions; we recommend the use of any BISCO adhesive.

- (3) Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing CORE-FLO DC.

- (4) a. If using a matrix band: place a matrix band around the prepared tooth. Place the end of the intra-oral lip directly against the preparation and dispense CORE-FLO DC. Light cure for 40 seconds.
- b. If using a core form: place the mixing tip directly into the core form and inject CORE-FLO DC. Place the loaded core form onto the prepared tooth and hold in place. Light cure for 40 seconds.

- (5) Prepare and finish the core in the usual manner.

2. POST CEMENTATION AND CORE BUILD-UP TECHNIQUE USING CORE-FLO DC

- (1) Isolate tooth and prepare the tooth according to the post manufacturer's instructions.
- (2) Apply an adhesive according to the manufacturer's instructions; we recommend the use of any BISCO adhesive. Prior to light curing, blot the canal dry with paper points until the paper point returns dry from the canal and gently remove the paper points.

- (3) Coat the surface of the post with an adhesive according to the manufacturer's instructions. This will increase the wettability of the post.

- (4) Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing CORE-FLO DC.

- (5) Using the root canal mixing tip inject CORE-FLO DC into the canal. Slowly withdraw the tip of the canal and withdraw the tip slowly while keeping the tip in the cement to minimize voids.

- (6) Coat the apical end of the post with CORE-FLO DC. Seat the post into the canal gently and maintain firm pressure for 30 seconds until the post is seated.
- (7) Continue to express CORE-FLO DC around the post and build-up the core to the desired height and shape.

- (8) Remove the excess CORE-FLO DC with a cure for 40 seconds by placing the light tip as close as possible to the post and core build-up.

- (9) Prepare and finish the core in the usual manner.

3. DENTIN REPLACEMENT MATERIAL

- (1) Isolate tooth and prepare in a conservative manner. Rinse and dry.

- (2) Apply an adhesive according to the manufacturer's instructions; we recommend the use of any BISCO adhesive.

- (3) Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing CORE-FLO DC.

- (4) Express CORE-FLO DC only to the level of the dentin-enameal junction.
- (5) Light cure the CORE-FLO DC for 10 seconds. Continue to firmly pack a light-cured composite, building incrementally until cavity preparation is filled. Light cure according to manufacturer's instructions.

- (6) Check the occlusion and continue with finishing and polishing procedures.

- (7) To prevent polymerization or clogging, the dual-syringe mixing tip should be left in place until the next application. Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If used intraorally, wipe mixing tip with disinfectant prior to storage.

- (8) **DISPOSAL:** Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

- (9) **STORAGE:** Store kit at room temperature (20°C/68°F - 25°C/77°F). See individual component labels for specific expiration dates.

- (10) **WARRANTY:** BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

- * CORE-FLO is a trademark of BISCO, Inc.

- Dual-Syringe Dispenser is distributed by BISCO, Inc.

- To Reorder Accessories:
 - X-81252P – Dual-Syringe Static Mixing Tips (30)
 - X-81257P – Dual-Syringe Mixing and Intraoral Tips (30)

- NOTICE:** Report any serious incident to BISCO and the competent authority in the member state in which the user and/or patient is established.

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