


BISCO


TheraCal PT[®]

Dual-Cured


Dual-Cured Resin-Modified Calcium Silicate *Pulpotomy Treatment*

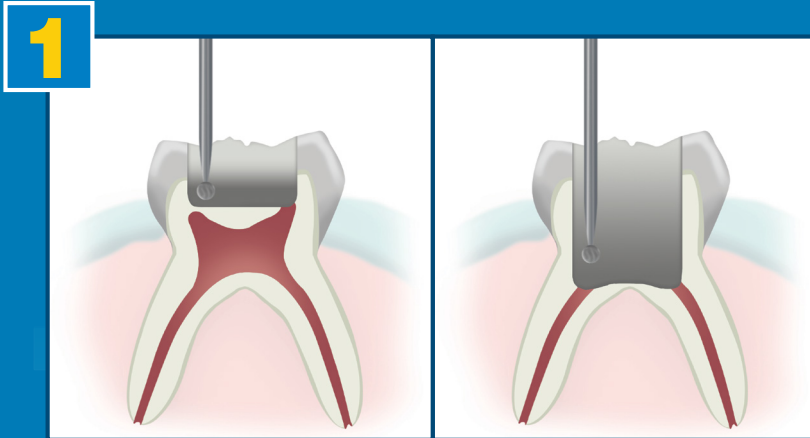


Perform pulpotomy treatment.

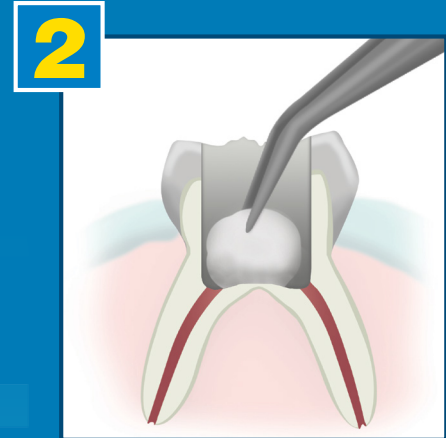

Achieve hemostasis.

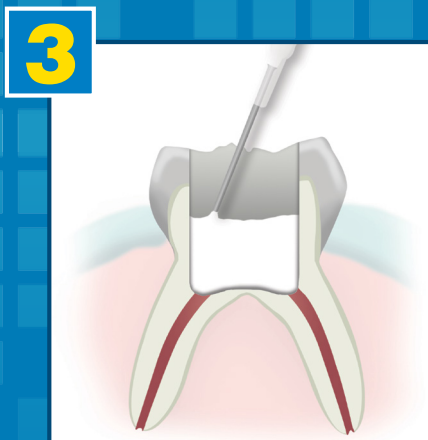


Place TheraCal PT directly in the pulp chamber, and ensure good adaptation to the cavity walls and margins.

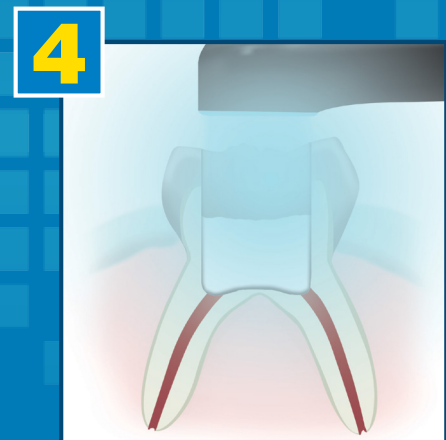


Light cure TheraCal PT for 10 seconds. Place desired adhesive, base, and/or restoration.



Rx Only

 BISCO, Inc. • 1100 W. Irving Park Road • Schaumburg, IL 60193 • U.S.A. • 1-847-534-6000 • www.bisco.com

 BISCO France • 208, allée de la Coudoulette • 13680 Lançon de Provence • France • 33-4-90-42-92-92



MC-4014TP