**PORCELAIN PRIMER****Pre-Hydrolyzed Silane Primer****Instructions for Use**

**BISCO**, Inc.  
1100 W. Irving Park Rd.  
Schaumburg, IL 60193

U.S.A.  
1-847-534-6000  
1-800-247-3368

IN-017R8  
Rev. 9/18

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

**PORCELAIN PRIMER\*****Pre-Hydrolyzed Silane Primer****GENERAL INFORMATION**

PORCELAIN PRIMER is a single component silane coupling agent in an alcohol and acetone base that is used to improve the adhesive bond between porcelain and composite. Silane is a dual function monomer consisting of a silanol group that reacts with the porcelain surface, and a methacrylate group that co-polymerizes with the resin matrix of the composite. Silane coupling agents are known to enhance the wettability of glass substrates by composite resins, increase physical, mechanical and chemical bonding of composite to porcelain, and yield a greater resistance to water attack at the bonding interface.

Prior to applying silane, the porcelain repair site must be etched with PORCELAIN ETCHANT\* (4% or 9.5% hydrofluoric acid). This process allows porcelain to become not only highly porous, but also highly receptive to the silane by creating reactive hydroxyl groups on the surface of the porcelain.

**Indications For Use:**

- A. Bonding Etched Porcelain/Lithium Disilicate Restorations
- B. Repair of Porcelain/Lithium Disilicate Restorations
- C. Repair of Porcelain-Fused-to-Metal
- D. Repair of Porcelain-Fused-to-Zirconia/Alumina

**Warnings:****Cautions:****Precautions:****INSTRUCTIONS FOR USE**

**A. Bonding Etched Porcelain/Lithium Disilicate Restorations:**

NOTE: If the laboratory has not etched the porcelain restoration, contact lab and/or ceramic manufacturer for proper etching instructions. Relisted BISCO products include 4% and 9.5% PORCELAIN ETCHANTS (Buffered Hydrofluoric Acid Gels).

**B. Repair of Porcelain/Lithium Disilicate Restorations:**

1. Brush on 2 thin coats of PORCELAIN PRIMER to the internal surface of the etched porcelain restoration and wait for 30 seconds to remove any salts. Rinse and dry thoroughly.

**C. Repair of Porcelain-Fused-to-Metal:**

1. Isolate the area to be repaired.

2. Remove the glaze and bevel (45°) the porcelain around the area to be repaired. Sandblast entire area or abrade with a coarse diamond bur. Rinse with water and air dry.

3. Place BARRIER GEL<sup>®</sup> on gingival tissue that may be exposed to protect areas where etching is not desired.

4. Apply PORCELAIN ETCHANT to the dry porcelain surface according to manufacturer's instructions. Continue to observe the etch site throughout the entire procedure. Suction the PORCELAIN ETCHANT, then rinse with a copious amount of water and air dry. The etched surface should appear dull and frosty.

5. Clean the etched porcelain by applying UNI-ETCH<sup>®</sup> and agitate for 30 seconds to remove any salts. Rinse and dry thoroughly.

6. Apply 1 thin coat of PORCELAIN PRIMER to the etched porcelain surface and allow to dwell for 30 seconds.

7. Apply a layer of PORCELAIN BONDING RESIN, air thin (light cure optional).

8. Complete the repair using a microhybrid composite (e.g. ELITE All-Purpose Body) and finish/polish.

**D. Repair of Porcelain-Fused-to-Zirconia/Alumina:**

1. Isolate the area to be repaired.

2. Remove the glaze and bevel (45°) the porcelain around the area to be repaired. Sandblast entire area or abrade with a coarse diamond bur. Rinse with water and air dry.

3. Place BARRIER GEL<sup>®</sup> on gingival tissue and metal that may be exposed to protect areas where etching is not desired.

4. Apply PORCELAIN ETCHANT to the dry porcelain surface according to manufacturer's instructions. Continue to observe the etch site throughout the entire procedure. Suction the PORCELAIN ETCHANT, then rinse with a copious amount of water and air dry. The etched surface should appear dull and frosty.

5. Clean the etched porcelain by applying UNI-ETCH<sup>®</sup> and agitate for 30 seconds to remove any salts. Rinse and dry thoroughly.

6. Apply 1 thin coat of PORCELAIN PRIMER to the etched porcelain surface and allow to dwell for 30 seconds.

7. Apply 1-2 coats of Z-PRIME<sup>™</sup> Plus to the exposed zirconia and alumina, air thin (light cure optional).

8. Apply a layer of PORCELAIN BONDING RESIN over porcelain and zirconia or alumina, air thin (light cure optional).

9. Complete the repair using a microhybrid composite (e.g. ELITE All-Purpose Body) and finish/polish.

**DISPOSAL:** Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

**STORAGE:** Store at room temperature (20°C/68°F - 25°C/77°F). Refer to individual component labels for specific expiration dates.

**WARRANTY:** BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

\*PORCELAIN PRIMER, PORCELAIN ETCHANT, PORCELAIN BONDING RESIN, BARRIER GEL and OPAQUER are manufactured by BISCO, Inc. UNI-ETCH is a registered trademark of BISCO, Inc.

\*\*ELITE and Z-PRIME are trademarks of BISCO, Inc.

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