


BISCO FluoroCal™

5% Sodium Fluoride Varnish with Tri-Calcium Phosphate


FluoroCal Technique




Clean the tooth surfaces (prophylaxis is not required). Open the blister pack by peeling back and removing foil lid, then remove the applicator brush.

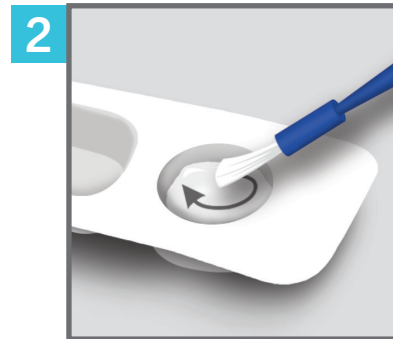

Thoroughly mix FluoroCal with the applicator brush for at least 5 seconds.

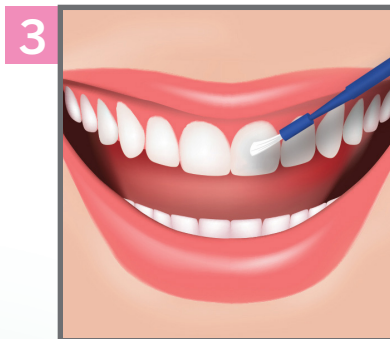

Apply FluoroCal evenly in a thin layer over treatment area.

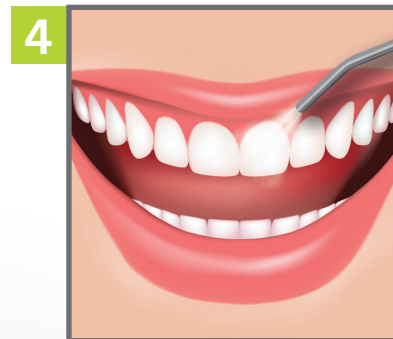

Dry the treated surfaces with air for 10 seconds. Patient should refrain from eating or drinking for at least 1 hour.



Rx Only

BISCO, Inc. • 1100 W. Irving Park Road • Schaumburg, IL 60193 • U.S.A. • 1-847-534-6000 • www.bisco.com

<https://www.bisco.com/symbols-glossary/>



MC-4116FC