

*Bisco*

CE 0459

***ORTHO Band Paste LC***

*Light-  
Cured*

*Fluoride Releasing Band Bonding System*

## Instructions for Use



BISCO, Inc.  
1100 W. Irving Park Rd.  
Schaumburg, IL 60193  
U.S.A.  
1-847-534-6000  
1-800-247-3368

IN-133R8  
Rev. 11/17

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

# ORTHO BAND PASTE LC

## Fluoride Releasing Band Bonding System

### GENERAL INFORMATION

**ORTHO BAND PASTE LC** is a light-cured orthodontic paste for band cementation. This hydrophilic system combines a resin base with a fluoride releasing agent. **ORTHO BAND PASTE LC** is available in a **BLUE** shade.

#### Indications For Use

Cementation of Orthodontic Bands

#### Cautions:

- Cross-contamination: Product may contain items that are designed for one time use. Dispose of used or contaminated unit-dose components and accessories. Do not clean, disinfect or reuse.

#### Precautions:

- Avoid contact with the skin; unpolymerized (meth)acrylate resins may cause skin sensitization in susceptible persons. In case of contact, wash skin with soap and water.
- Avoid irritating the gingival tissue since slight bleeding may cause contamination and could affect bonding quality.
- Refrigeration is necessary when not in use (2°C/36°F - 8°C/46°F).
- Allow refrigerated **ORTHO BAND PASTE LC** to reach room temperature before use
- Safety data sheet available on request.
- Safety data sheet available at [www.Bisco.com](http://www.Bisco.com).

### INSTRUCTIONS FOR USE

**NOTE: Allow material to reach room temperature prior to use.**

1. Prophylaxis - Thoroughly clean the teeth to be bonded with pumice in a rubber cup or bristle brush. Carefully avoid irritating the gingival tissue since slight bleeding may cause contamination and could affect bonding quality.
2. Rinse thoroughly with plenty of water. Isolate the teeth and dry with the air syringe.
3. Apply **ORTHO BAND PASTE LC** on the internal side of the band.
4. Seat the band on the tooth to the desired position.
5. Light cure the paste from the occlusal surface for 10 seconds. Remove the excess material (it should come off easily).
6. Light cure from the occlusal surface for an additional 20 seconds or until completely cured.

#### DEBONDING PROCEDURE:

1. Remove bonded bands/crowns from the buccal or lingual side using band/crown removing pliers.
2. Remove any residual material with adhesive removing pliers, scaler or rotary instrument.

**HYGIENE:** Re-cap syringes with the luer lock cap. Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended.

**DISPOSAL:** Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

**STORAGE:** Refrigerate when not in use (overnight) (2°C/36°F - 8°C/46°F). Refer to individual component labels for specific expiration dates.

**WARRANTY:** BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

BISCO, INC.  
1100 W. Irving Park Rd.  
Schaumburg, IL 60193  
U.S.A.  
1-800-247-3368  
1-847-534-6000  
[www.bisco.com](http://www.bisco.com)



BISICO France  
120, allée de la Coudoulette  
13680 Lançon De Provence  
France  
33-4-90-42-92-92