

Bisco

CE

NB Instrument Kit

Composite Placement Instruments

Instructions for Use



BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-847-534-6000
1-800-247-3368

IN-222R1
Rev. 12/17

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NB Instrument* Kit Composite Placement Instruments

Inspect instrument for damage prior to use. Do not use if there is any damage.

Clean instrument as soon as possible after use: Instruments can be washed by hand or ultrasonic or automatic washer using a non-corrosive, non-suds, neutral detergent. Do not use abrasives.

Sterilize before first use and between patients: Autoclave 20 minutes at 121°C/250°F (15 psi)

* NB Instruments are manufactured by BISCO, Inc.

BISCO, INC.
1100 W. Irving Park Rd.
Schaumburg, IL 60193
U.S.A.
1-847-534-6000
1-800-247-3368
www.bisco.com

 BISCO France
120, allée da la Coudoulette
13680 Lançon de Provence
France
33-4-90-42-92-92