


ZirClean™




1. Try in (to check occlusion and fitting).
2. After try-in, thoroughly rinse the restoration with water spray and dry with oil-free air.

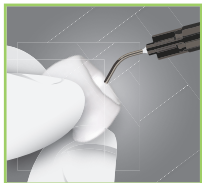

3. Cover all bonded surfaces of the restoration with a layer of ZirClean. Tips are single use only; make sure to dispose of them properly before using material again.
4. Allow 20 seconds for the cleaning action of ZirClean to take effect, then thoroughly rinse with water spray and dry with oil-free air.

* Sandblast internal surface of restoration if necessary.

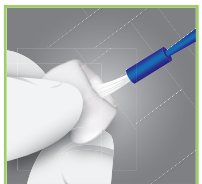

5. Prime the bonding surface of the restoration with a suitable primer (such as Z-PRIME™ Plus* or PORCELAIN PRIMER*) according to manufacturer's instructions.



Consult
Instructions
For Use

INTC-075R1

The logo for ZirClean features the word "ZirClean" in a blue, sans-serif font. A green swoosh arches over the letters "ir" and "C". A small trademark symbol (TM) is positioned to the upper right of the word.

ZirClean™