

Bisco

Pro-V[®] IDS KIT

Provisional Restorative System

Provisorisches Restaurationssystem

Système de restauration provisoire

Sistema Restaurador Provisional

Sistema per restauri provvisori

Provisoriskt restaurationssystem

Provisorisk restaureringssystem

Väliaikainen paikkausmenetelmä

Tijdelijk restauratiesysteem

Provisorisk Restaurasjons System

Sistema de Restauração Provisória

Σύστημα για προσωρινές αποκαταστάσεις

Временная Реставрационная Система

Immediate Dentin Sealing Kit REF M-10120K

CONTENTS:

1 Bottle ALL-BOND UNIVERSAL[®] (6ml), 1 Syringe PRO-V FILL[®] (4g),
1 Syringe AELITEFLO[™] LV (1.5g), 1 Bottle PRO-V COAT[®] (4ml), 1 Bottle
CAVITY CLEANSER[™] (7ml), 1 Syringe TheraCal LC[®] (1g), Accessories,
Instructions



Single Use
Accessories




For Dental
Use Only



Caution



Consult
Instructions
For Use

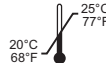

Recommended
Storage Temp.



Flammable



Irritant



Sensitizer


Manufacturer

BISCO, Inc.


**1100 W. Irving Park Rd.
Schaumburg, IL 60193
U.S.A.**


Phone


847-534-6000

INST-109
Rev. 2/15


IMMEDIATE DENTIN SEALING USING ALL-BOND UNIVERSAL*

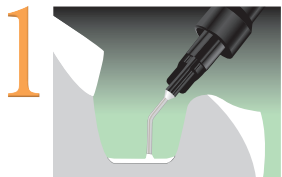


1 Isolate and prep tooth per inlay/onlay preparation guidelines. If pulp is exposed; achieve hemostasis, apply 1 mm layer of TheraCal LC directly on and around the exposed pulp, then light cure for 20 seconds. If prep is near the pulp, apply a 1 mm layer of TheraCal LC as a liner and light cure for 20 seconds.

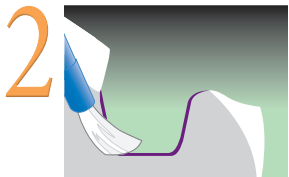


2 Apply CAVITY CLEANSER, let dwell for 30 seconds. Suction and blot excess.

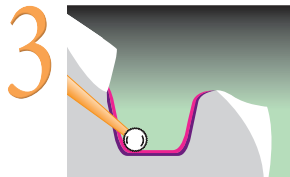


3 Apply two separate coats of ALL-BOND UNIVERSAL, scrubbing the preparation with a microbrush for 10-15 seconds per coat. Do not light cure between coats. Evaporate solvent by thoroughly air-drying with an air syringe for at least 10 seconds. Light cure for 10 seconds.

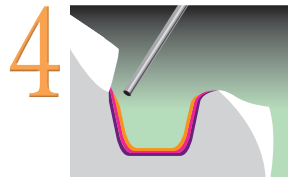


4 Apply a THIN layer of AELITEFLO LV across pulpal floor and light cure for 10 seconds; if necessary, block proximal wall undercuts with flowable and light cure for 20 seconds.

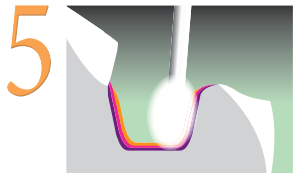


5 Remove air inhibited layer with an alcohol pellet.

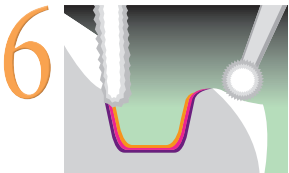

6 Use finishing bur to re-prep and smooth occlusal enamel margins. Take final impression.

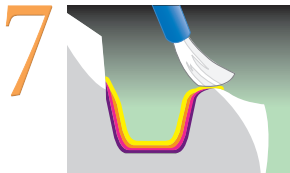

7 Apply PRO-V COAT to lubricate sealed dentin. Gently air dry for 10-15 seconds.

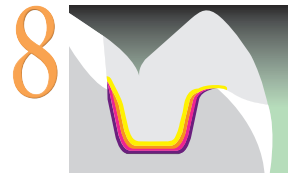

8 Build temporary in increments with PRO-V FILL in 2-3 mm increments. Light cure each increment for 10 seconds. After final adjustment, light cure for 20 seconds.

* Refer to instructions for use for detailed technique, including IDS technique for Crown preparations.